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**Application:** Make a 7.5% application solution of the VIREX BASIC Conc. SD 200 Disinfectant and Cleaner. Add the 150ml VIREX BASIC Conc. SD 200 sachets (bottle) to the bucket and add 1.85 liters of tap water. **Note:** If the concentration is different, mix VIREX Basic Conc. SD 200 with tap water to give 2 liters of application solution. Then place the complete hospyWipes cloth rail in the dispenser bucket. Pull the beginning of the roll (in the middle of the roll) in through the application opening in the lid and close it tightly with the bucket. After 20 minutes, the 2 liters of working solution are completely absorbed. Now you can set the dispenser bucket (upside down) in the wall holder. The wipes are now tear-off and should be removed individually from the application opening. Only use the hospyWipes cloths as long as they are soaked with a disinfectant solution. Dry cloths have no disinfecting effect. Wipe surfaces and objects carefully. Observe contact time!

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#### Fleece Cloths for Soaking in Disinfectant and Cleaning Agent

**Product Description:** Dry wipes for soaking in surface disinfectant or cleaning agent for fast and uncomplicated disinfection and cleaning of surfaces and medical products. The cloths are made from qualitative high-grade, fluff-free fleece material. The wipes have been developed especially for surface disinfection. They are excellent for applying the disinfectant solution and ensure a very good wetting of the surface to be disinfected. **Field of Application:** Fast, uncomplicated and economical disinfection and cleaning of surfaces, medical equipment and medical products. **Composition:** 1 roll is made up of 600 dry wipes of high-quality, fluff-free viscose perforated cloth. **Important Note:** Special labels which facilitate the documentation of batch, date of use and use-by-date of the corresponding disinfectant or cleaning agent, are additionally available. **Using VIREX BASIC Konz. SD 200 please stick attached label on a free side of the bucket.**

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Prisman GmbH